

Test With Changes in Collection - Quest 12/17 Go-live

Below is a crosswalk of the testing scheduled to go-live on 12/17/2025 that have changes to their collection containers. Links to the Quest lab test directory are provided below. Details on these tests will be updated in our lab test directories on 12/17.

[Laboratory Catalogs | Corewell Health](#)

Current EPIC Code	FUTURE Quest EPIC Order	Collection Detail	Preferred Collection	Container Changes
COENZYME Q10, REDUCED AND TOTAL, PLASMA [LAB3151]	COENZYME Q10 [LAB12320757]	5 ML GOLD TOP SST R [1230300004]	Gold Top serum	No longer green top / plasma
LYSOZYME (MURAMIDASE), PLASMA [LAB1230807]	LYSOZYME (MURAMIDASE) [LAB1232018]	5 ML GOLD TOP SST R [1230300004]	Gold Top serum	No longer Lavender / plasma
PARATHYROID HORMONE-RELATED PEPTIDE (PTH-RP), PLASMA [LAB7040]	PTH-RELATED PROTEIN (PTH-RP) [LAB1232020]	6 ML NA HEPARIN A [1230300037]	Sodium heparin plasma	No longer Pink/Lavender EDTA Plasma
RIBOFLAVIN (VITAMIN B2), PLASMA [LAB3216]	VITAMIN B2 (RIBOFLAVIN) [LAB1232037]	6 ML RED TOP R [1230300000]	Red top serum or lavender plasma; Protect from light	No longer green top / plasma
SELENIUM, SERUM [LAB579]	SELENEIUM [LAB1232265]	ROYAL BLUE TRACE ELEMENTS [1230300047]	Royal blue trace element serum or plasma in trace element-free transfer tube	No longer Royal Blue no additive
SEROTONIN, WHOLE BLOOD [LAB1231463]	SEROTONIN, BLOOD [LAB1232209]	MISCELLANEOUS [1230300177]- Quest Serotonin Kit	Quest Serotonin Kit; 4 mL whole blood	No longer Lav-top EDTA poured over to Ascorbic Acid tube.
PRIMIDONE AND PHENOBARBITAL, SERUM [LAB485]	PRIMIDONE AND PHENOBARBITAL [LAB1232238]	6 ML RED TOP R [1230300000]	Red top serum or green / lavender plasma	No longer Gold Top SST
24-Hour Urine Tests				
Current EPIC Code	FUTURE Quest EPIC Order	Collection Detail	Preferred Collection	Container Changes
ALDOSTERONE, 24 HOUR URINE [LAB354]	ALDOSTERONE, 24 HOUR URINE [LAB1232167]	No preservative needed	24-hour urine, refrigerated	NO Boric Acid prior to collection
HEAVY METAL, CREATININE RATIO, WITH REFLEX, RANDOM, URINE [LAB1230733]	HEAVY METALS PANEL 4, URINE (CSA) [LAB1232141]	No preservative needed	24-hour urine, refrigerated	NO 6N Hydrochloric Acid prior to collection